

# IMPLANT

## Torque & Driver Settings

MANUFACTURER	IMPLANT	TORQUE NCM	IMPLANT DRIVERS
<b>BIOHORIZONS</b>			
	Internal 3.5, 4.5, 5.7	30 Ncm	0.050" (1.25 mm) Hex
	Tapered Internal 3.0, 3.5, 4.5, 5.7	30 Ncm	0.050" (1.25 mm) Hex
	External Implant 3.5	30 Ncm	0.050" (1.25 mm) Hex
	External Implant 4.0, 5.0	30 Ncm	0.050" (1.25 mm) Hex
<b>BIOMET 3i</b>			
	Certain™ 3.25, 4.0, 4/3, 5.0, 5/4, 6.0	20 Ncm	0.048" (1.22 mm) Hex
	Certain™ Prevail™ 3/4/3 (3.4), 4/5/4, 5/6/5, 4/3, 5/4, 6/5	20 Ncm	0.048" (1.22 mm) Hex
	Certain™ XP 4/5, 5/6	20 Ncm	0.048" (1.22 mm) Hex
	Miniplant™ 3.25	20 Ncm	0.048" (1.22 mm) Hex
	Osseotite™ 3.75, 4.0, 5.0, 6.0	20 Ncm	0.048" (1.22 mm) Hex
	Osseotite™ XP 4/5, 5/6	20 Ncm	0.048" (1.22 mm) Hex
<b>DENTSPLY</b>			
	ANKYLOS® C/X - C/3.5, 4.5, 5.5, 7.0 (non-indexed)	15 Ncm	0.039" (1.00 mm) Hex
	ANKYLOS® C/X - /X 3.5, 4.5, 5.5, 7.0 (indexed)	15 Ncm	0.039" (1.00 mm) Hex
	OsseoSpeed EV 3.0 - Green	25 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ EV 3.6 - Purple	25 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ EV 4.2 - Yellow	25 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ EV 4.8 - Blue	25 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ EV 5.4 - Brown	25 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ Profile EV 4.2 Yellow	25 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ Profile EV 4.8 Blue	25 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ / OsseoSpeed TX 3.0- Yellow	15 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ / OsseoSpeed TX 3.5, 4.0 - Aqua	20 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ / OsseoSpeed TX 4.5, 5.0 - Lilac	25 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ 4.0 - Blue (Previous)	25 Ncm	0.050" (1.25 mm) Hex
	OsseoSpeed™ TX Profile 4.5, 5.0	25 Ncm	0.050" (1.25 mm) Hex
	XIVE® S 3.0	24 Ncm	0.048" (1.22 mm) Hex
	XIVE® S 3.4,3.8, 4.5, 5.5	24 Ncm	0.048" (1.22 mm) Hex
	FRIALIT™ 3.4, 3.8, 4.5, 5.5	24 Ncm	0.048" (1.22 mm) Hex

**We Are Here To Help**



**P: 417.881.8572**  
**P: 800.462.3669**  
[edmondsdentalprosthetics.com](http://edmondsdentalprosthetics.com)

MANUFACTURER	IMPLANT	TORQUE NCM	IMPLANT DRIVERS
<b>KEYSTONE</b>			
	PrimaConnex™ SD 3.3, 3.5, Genesis™ 3.8	30 Ncm	Keystone Quad Driver
	PrimaConnex™ RD 4.0, 4.1, Genesis™ 4.5	30 Ncm	Keystone Quad Driver
	PrimaConnex™ WD 5.0, Genesis™ 5.5, 6.5	30 Ncm	Keystone Quad Driver
	Restore™ SD 3.3, 3.4	30 Ncm	0.048" (1.22 mm) Hex
	Restore™ RD 3.75, 4.0	30 Ncm	0.048" (1.22 mm) Hex
	Restore™ WD 5.0, 5.5, 6.0	30 Ncm	0.048" (1.22 mm) Hex
	Mutli-Unit Abutment	30 Ncm	Prima Quad Driver
	Chimneys (Temp Cylinders) Conversion Cylinders (Dentures & Full Arch)	30 Ncm	Prima Quad Driver
<b>NEODENT</b>			
	CM Abutment	35 Ncm	1.22 mm Hex
	GM Exact Abutment	20 Ncm	Neo Driver Connection
<b>NOBEL BIOCARE</b>			
	Angulated Screw Channel	35 Ncm	Nobel Biocare® Omnigrip
	NobelActive 3.0	15 Ncm	Nobel Biocare® Unigrip
	NobelActive™/NobelReplace™ Conical Connection NP 3.5	35 Ncm	Nobel Biocare® Unigrip
	NobelActive™/Nobel Replace™ Conical Connection RP 4.3, 5.0	35 Ncm	Nobel Biocare® Unigrip
	NobelReplace™ NP 3.5	35 Ncm	Nobel Biocare® Unigrip
	NobelReplace™ RP 4.3	35 Ncm	Nobel Biocare® Unigrip
	NobelReplace WP 5.0	35 Ncm	Nobel Biocare® Unigrip
	NobelReplace™ WP 6.0	35 Ncm	Nobel Biocare® Unigrip
	Branemark System™ NP 3.3	35 Ncm	Nobel Biocare® Unigrip
	Branemark System™ RP 3.75, 4.0	35 Ncm	Nobel Biocare® Unigrip
	Branemark System™ WP 5.0, 5.5	35 Ncm	Nobel Biocare® Unigrip
<b>STRAUMANN</b>			
	Bone Level 3.3 NC (Narrow Crossfit™)	35 Ncm	SCS Screwdriver
	Bone Level 4.1, 4.8 RC (Regular CrossFit™)	35 Ncm	SCS Screwdriver
	Standard/Standard Plus 4.8 RN (Regular Neck)	35 Ncm	SCS Screwdriver
	Standard/Standard Plus 4.8 WN (Wide Neck)	35 Ncm	SCS Screwdriver
	Standard Plus 3.3 NN (Narrow Neck)	35 Ncm	SCS Screwdriver
<b>ZIMMER BIOMET</b>			
	Tapered Screw-Vent 3.5, 4.5, 5.7/Tapered Screw-Vent 3.5, 4.5, 5.7 Friction Fit	30 Ncm	0.050" (1.25 mm) Hex
	Screw-Vent 3.3, 3.7, 4.5/Screw-Vent 3.3, 3.7, 4.5 friction fit	30 Ncm	0.050" (1.25 mm) Hex
	SwissPlus 4.8	30 Ncm	0.050" (1.25 mm) Hex

**We Are Here To Help**



**P: 417.881.8572**  
**P: 800.462.3669**  
**edmondsdentalprosthetics.com**